

# KometaBio Dentin Protocol

## Accelerated Dentin Graft

Procedure for partial demineralization of a pre-ground KometaBio Dentin Graft.

	Step 1	Step 2	Step 3
			
Liquid	(RED Cap) Cleanser	(BLUE Cap) EDTA	(GREEN Cap) PBS Wash
Procedure	Cover graft for 5 minutes with solution and dehydrate with sterile gauze.	Cover graft for 2 minutes with solution and dehydrate with sterile gauze.	Cover graft with solution, mix with a sterile spatula or swirl and dehydrate with a sterile gauze. Repeat TWICE.
Duration	5 minutes	2 minutes	~1 minutes

**Note:** The above procedure will result in 20um of demineralized dentin surface exposure to induce osteogenic activity of dentin.

Ivy Cottage, Bridgnorth Road,  
Stourton, DY7 5BH

M. +44 (0) 7800 990 131  
E. [ask@regen-store.co.uk](mailto:ask@regen-store.co.uk)  
W. [www.regen-store.co.uk](http://www.regen-store.co.uk)

Company registration number 10714405



Incorporating Ariston Dental UK