


# TISSUE HARVESTING TOOL

## LESS TRAUMA – MORE CONTROL – SINGLE ACTION

Sterile single-use tool allows one-handed harvesting of a palatal soft tissue graft.

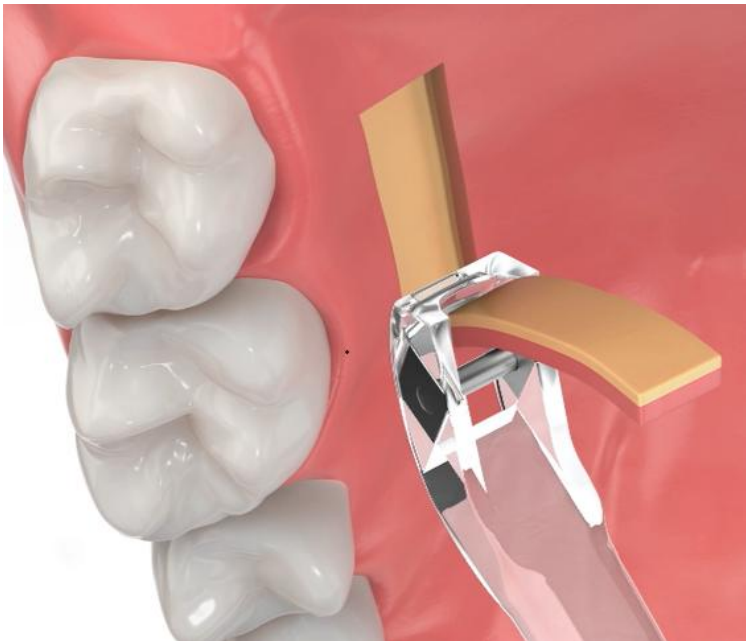

## HARVESTED TISSUE DIMENSIONS

### Thickness

1.5 – 2.5 mm based upon handle angle.

### Width

5.5 mm fixed by device head.

## INSTRUCTIONS FOR USE

- Anaesthetise donor site on the maxillary palate aspect.
- Place harvesting tool against the vertical wall of the maxillary posterior palatal region and apply gentle tissue pressure whilst drawing the instrument in a posterior-to-anterior direction.
- Prepare either the epithelial layer or underlying connective tissue.

## FIXATION | CLOSURE | PROTECTION

**Recipient site:** PGCL absorbing mono sutures or Periacryl Adhesive.

**Donor Site:** Periacryl or protect with Ora-Aid self-adhering dressing.

## PRESENTATION AND ITEM CODE

Two sterile harvesting tools per box. Code: 3500T15.

Ivy Cottage, Bridgnorth Road,  
Stourton, DY7 5BH

M. +44 (0) 7800 990 131  
E. ask@regen-store.co.uk  
W. www.regen-store.co.uk

Company registration number 10714405.



Incorporating Ariston Dental UK